

	<b>Product Name : Baby Weighing Scale</b>	<b>Doc no: 73-00-001</b>
	<b>Product Model No : nice 1000</b>	
	<b>Trouble Shooting</b>	

## 1. General System Failure

No display on the control panel	Check A/C Supply Voltage
	Check if Power Cord is disconnected
	Check Fuse in the left side
	Otherwise, Contact nice Neötech
Weight error	Check whether the pan is fixed properly in the machine
	Check the pan if any damages in the 4 knob
	Otherwise, Contact nice Neötech
Bubble indicator not in the centre	Check the table where the machine is placed; Ensure that the table has no movements.
	Adjust the knob at down of the machine by using the 10 / 11 spanner
	Otherwise, Contact nice Neötech